



# B.BRAUN REGIONAL ANAESTHESIA PORTFOLIO PRODUCT CATALOGUE

# Reach your Targets quicker


Thanks to Regional Anaesthesia

## OPTIMIZE PROCESSES

Regional anaesthesia is an essential part of pain therapy. Across several interventions, it offers clear advantages when compared to general anaesthesia. <sup>1</sup> For regional anaesthesia to become an important part of the peri-operative process, there are some prerequisites to consider, like the relevant indications, skilled personnel and the necessary internal processes at the hospital.

The rapid growth of use of regional anaesthesia and the numerous publications on the topic are a testament to how regional anaesthesia can have a positive impact on patient outcome as well as on efficiencies for the hospital.<sup>2,3</sup>

## WITH REGIONAL ANAESTHESIA

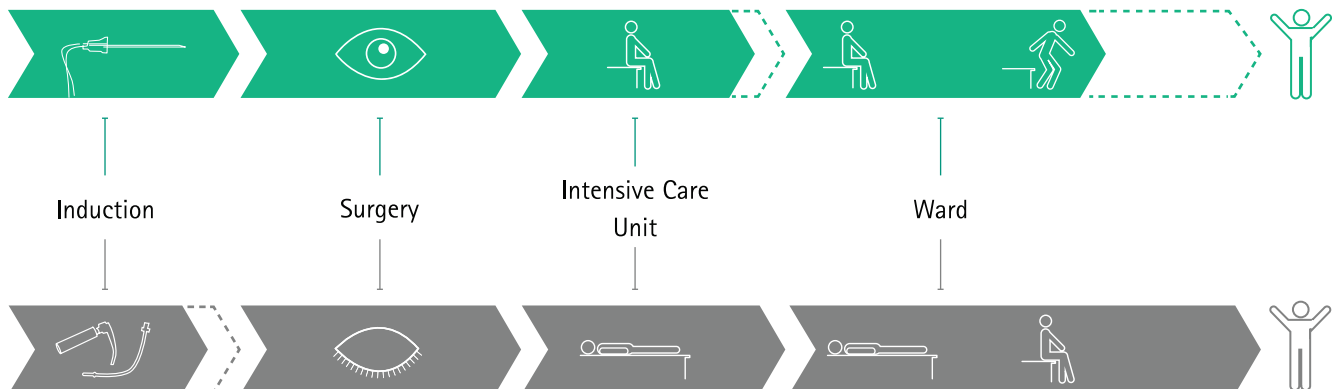

## WITHOUT REGIONAL ANAESTHESIA


### COMPARING TWO POTENTIAL INTERVENTIONS WITH AND WITHOUT REGIONAL ANAESTHESIA:

Looking holistically across the patient pathway, the establishment of regional anaesthesia processes appears to be a favourable investment. When regional anaesthesia is used, patients see better outcomes, faster mobilization and consequently earlier discharge.<sup>4,5</sup>

# Content

## SPINAL ANAESTHESIA


Spinocan® | Pencan® | Atraucan® | Spinal Introducer Needle

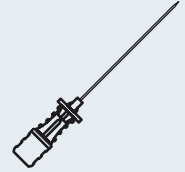

SPINAL ANAESTHESIA

## EPIDURAL ANAESTHESIA


Perican® | Perican® Paed | Epican® Paed  
Perifix® Epidural Catheter | Accessories

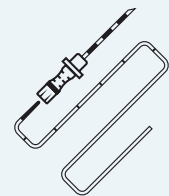

EPIDURAL ANAESTHESIA

## COMBINED SPINAL & EPIDURAL ANAESTHESIA


Espocan® Sets

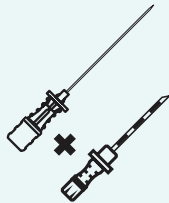

COMBINED SPINAL AND  
EPIDURAL ANAESTHESIA


## PERIPHERAL NERVE BLOCKS

Stimuplex® A | Stimuplex® Ultra 360® | Ultraplex® 360  
Contiplex® C | Contiplex® S Ultra 360® | Contiplex® Tuohy Ultra 360®

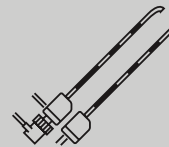

PERIPHERAL NERVE  
BLOCKS

## ULTRASOUND GUIDANCE

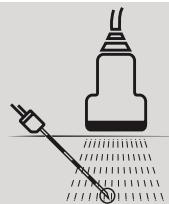

ULTRASOUND  
GUIDANCE

# Pain Therapy with Regional Anaesthesia

Supported by B. Braun's Product Portfolio



## B. BRAUN – A COMPETENT PARTNER

As one of the world's major providers in the area of regional anesthesia, B. Braun supports its customers with innovative products and services.


We strive to accompany our customers through the entire patient pathway helping to optimize their processes and providing products adapted to day-to-day clinical needs.

## SYSTEM PARTNER IN REGIONAL ANAESTHESIA


B. Braun, as a true system partner, helps to stay on top of evolving standards and their realization across multiple stakeholders, e. g. nurses, pharmacists and anaesthetists. With access to our comprehensive product portfolio in pain therapy, and through constructive dialogue, our partners can benefit from B. Braun's know-how and expertise.

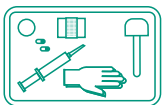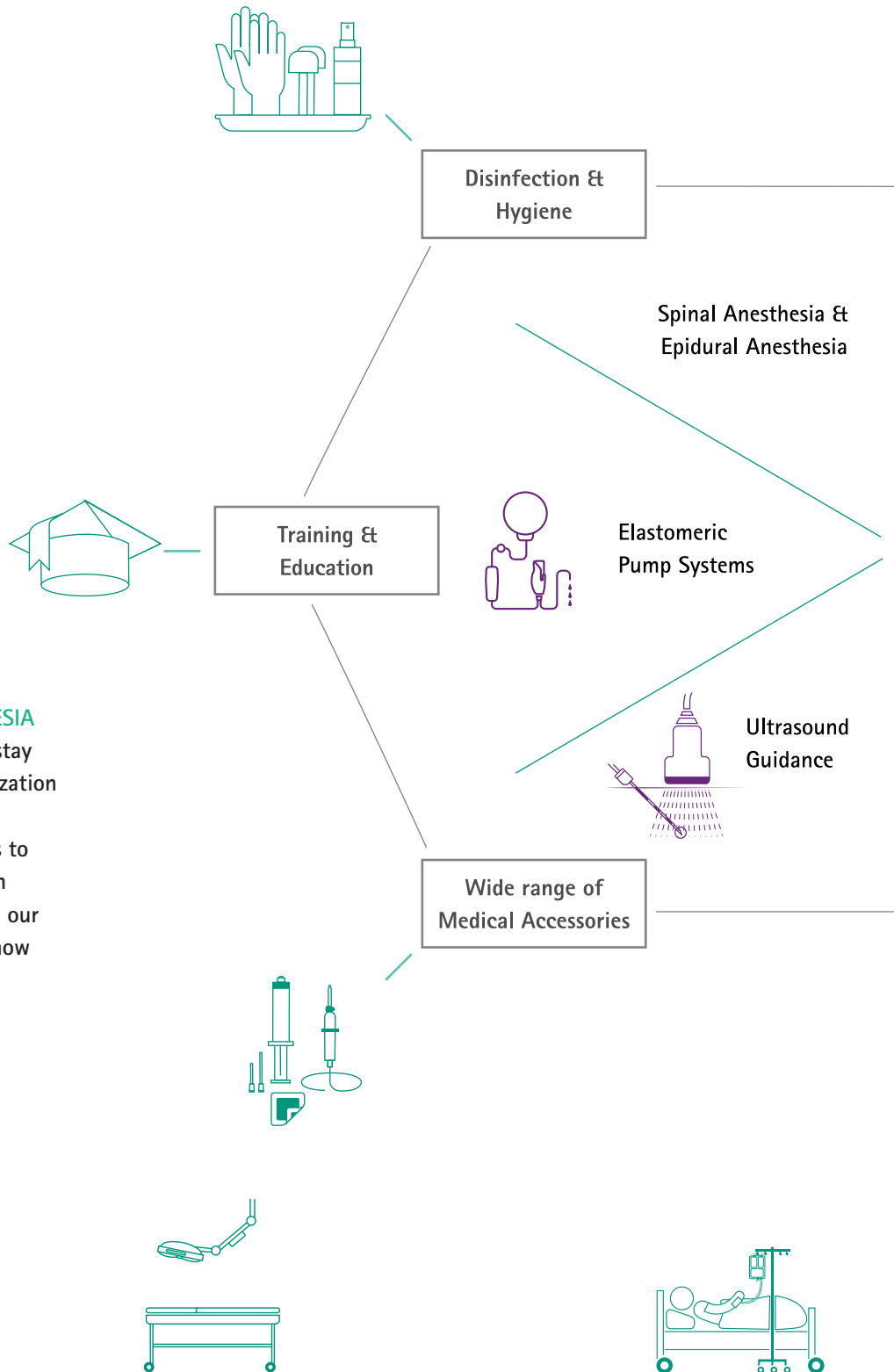



Introduction

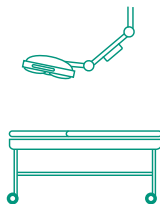

Surgery

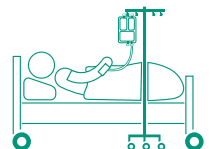



Recovery Room

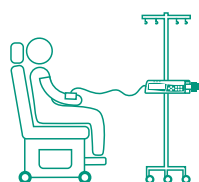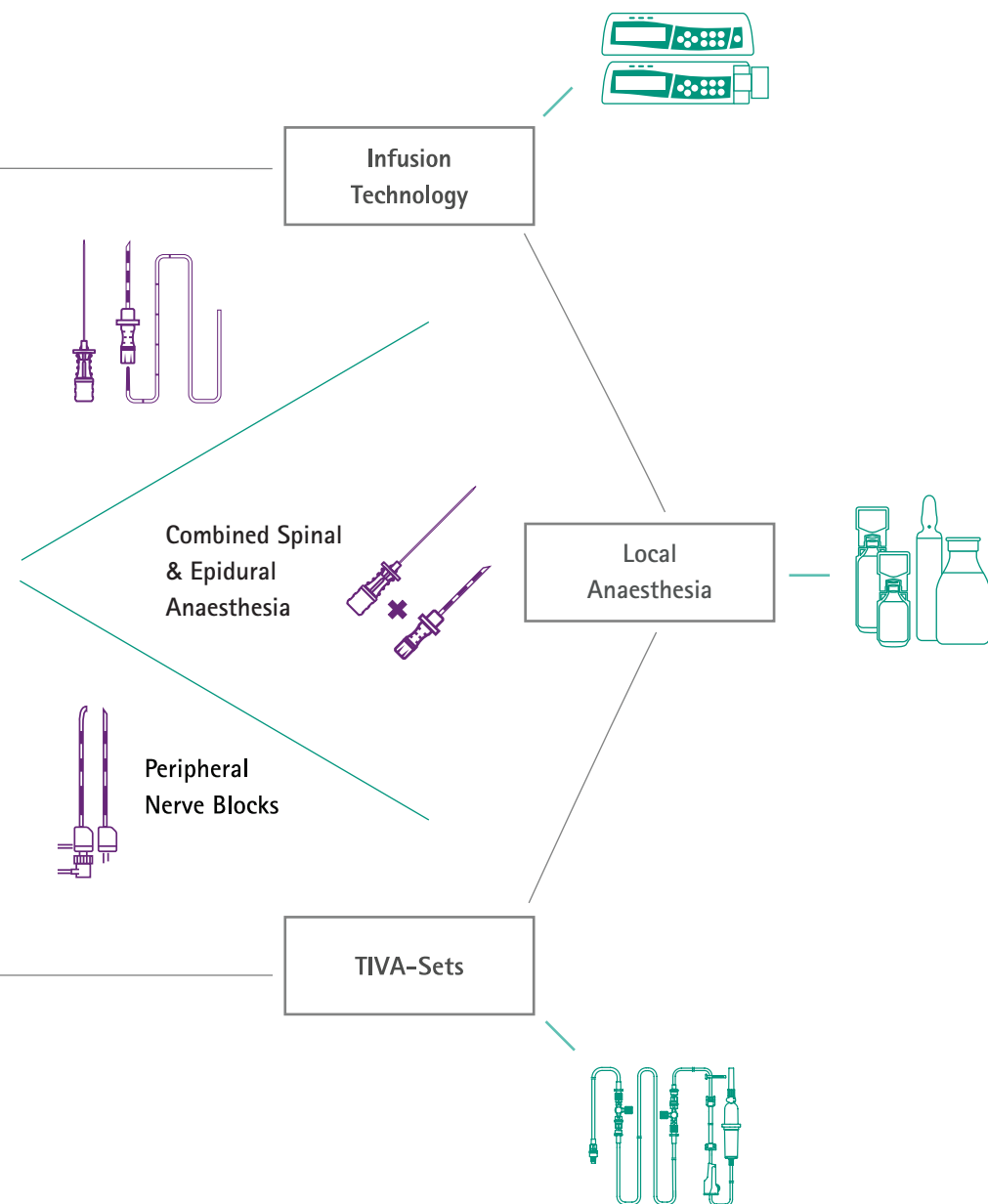






Due to the combination of different procedures, a broad regional anaesthesia spectrum is created which covers all patient areas.

# SPINAL ANAESTHESIA

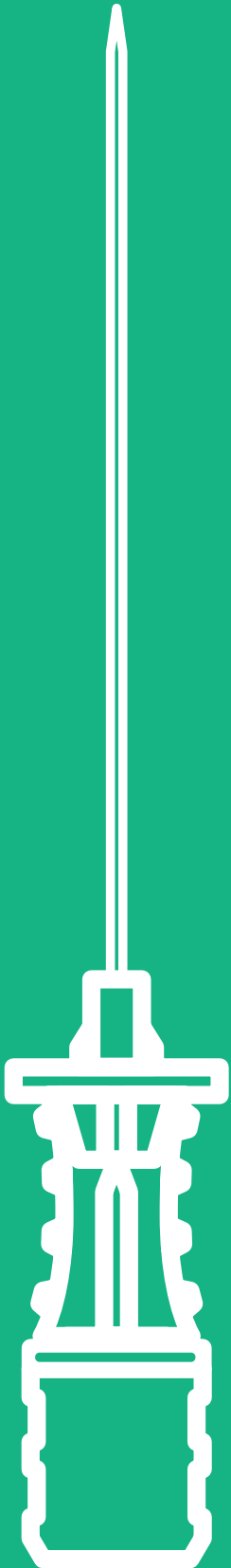

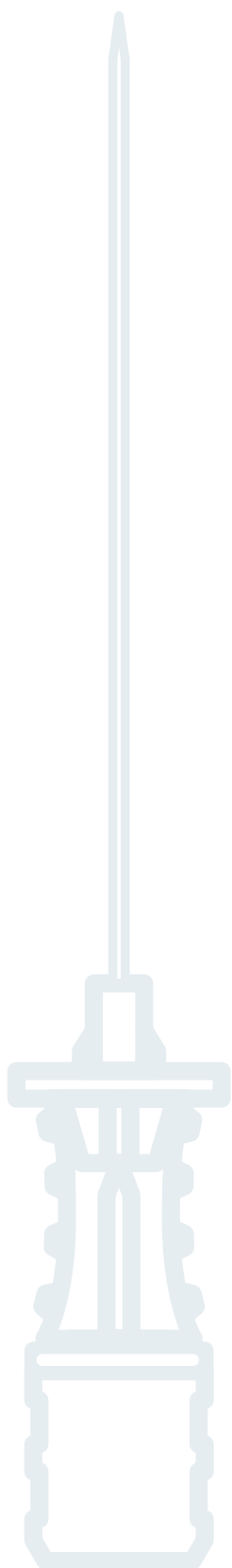

A comprehensive portfolio of  
needles in different bevel types  
for spinal anaesthesia.

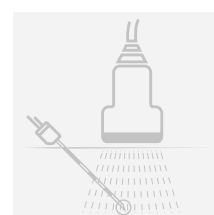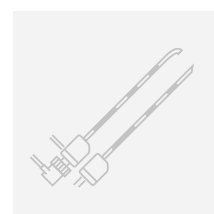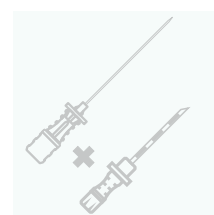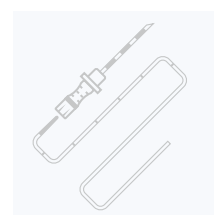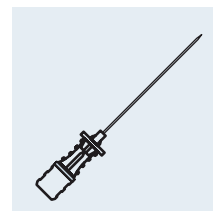

# Products for Spinal Anaesthesia

Quality and Practicality



**Ergonomic hub design** for an individual and safe handling position, better control of the needle and easy verification of bevel eye orientation.<sup>7</sup>

Thanks to our years of

## EXPERIENCE


in the field, B.Braun offers spinal needles with a range of bevel types.

Spinocan® and Pencan® are well accepted for their

## INNOVATIVE


needle tip designs.



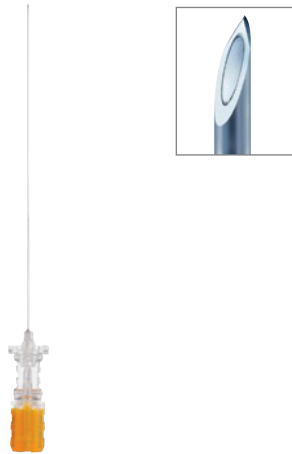

#### Spinocan® with Quincke Bevel

- Sharp bevel for low puncture force<sup>7</sup>
- Fitted stylet: to reduce the risk of tissue coring and potential epidermoid tumor development
- The spinal needle hub supports identification of CSF flashback

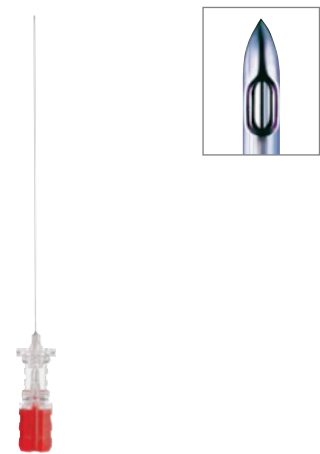

#### Pencan® with Pencil Point Bevel

- Pencil point tip and opening design allows avoidance of incomplete blocks and helps reduce PDPH<sup>8</sup>
- Minimal intrathecal penetration depth<sup>7</sup>
- Identification of tissue structures due to detectable dura click<sup>7,9</sup>
- The spinal needle hub supports identification of CSF flashback



#### Atraucan® with Two-Zone Bevel

- Low puncture force as well as minimal puncture trauma<sup>10</sup>
- Reduced risk of postdural puncture headache (PDPH)<sup>10</sup>

























#### Spinal Introducer Needle

- Introducer needles providing needle guidance for the spinal needle during puncture<sup>7</sup>











# Portfolio Overview
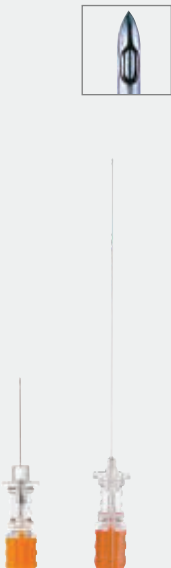
## SPINAL NEEDLES

Spinocan®	Color code	Size (G)/ Length (inch)	ø / Length (mm)	Units per box	Code no. (REF)	
		29 x 3 1/2"	0.35 x 88*	25	4501900	
		27 x 3 1/2"	0.40 x 88		4503902	
		26 x 3 1/2"	0.45 x 88		4502906	
		25 x 3"	0.55 x 75		4505751	
		25 x 3 1/2"	0.55 x 88		4505905	
		22 x 1 1/2"	0.70 x 40		4507401	
		22 x 3"	0.70 x 75		4507754	
		22 x 3 1/2"	0.70 x 88		4507908	
		20 x 3"	0.90 x 75		4509757	
		20 x 3 1/2"	0.90 x 88		4509900	
		19 x 1 1/2"	1.10 x 40		4501144	
		19 x 3 1/2"	1.10 x 88		4501195	
		18 x 3"	1.30 x 75		4501373	
		18 x 3 1/2"	1.30 x 88		4501390	
	Long Length					
		26 x 4 3/4"	0.45 x 120		4504917	
		25 x 4 3/4"	0.55 x 120		4505913	
		22 x 4 3/4"	0.70 x 120		4506090	




Atraucan®	Color code	Size (G)/ Length (inch)	ø/Length (mm)	Units per box	Code no. (REF)
		26 x 3 1/2"	0.47 x 88	25	4504739
	Atraucan® Paed				
		26 x 2"	0.47 x 50	25	4504763

## SPINAL NEEDLES

Pencan®	Color code	Size (G) / Length (inch)	ø / Length (mm)	Units per box	Code no. (REF)
		27 x 3 ½"	0.40 x 88	25	4502027
					4502051
		25 x 3 ½"	0.50 x 88		4502019
					4502043*
		22 x 3 ½"	0.70 x 88		4502035
	Long Length				
		27 x 4"	0.40 x 103	25	4502124*
		27 x 4 ¾"	0.40 x 120		4502132
		25 x 4"	0.50 x 103		4502116*
	Pencan® Paed				
	27 x 2"	0.40 x 50	25	4502175	
	25 x 1"	0.50 x 25		<b>4502167</b>	
	25 x 2"	0.50 x 50		4502159	

\*with Spinal Introducer Needle (Luer)


## INTRODUCER

Spinal Introducer Needles (for Spinocan® and Pencan®)	Color code	Size (G) / Length (inch)	ø / Length (mm)	Units per box	Code no. (REF)
	<b>For Spinal Needles 29 G and 27 G</b>				
		22 x 1 3/8"	0.70 x 35	25	4500059*
	<b>For Spinal Needles 26 G and 25 G</b>				
		20 x 1 3/8"	0.90 x 35	25	4505000*







\*Introducer in Luer



# EPIDURAL ANAESTHESIA

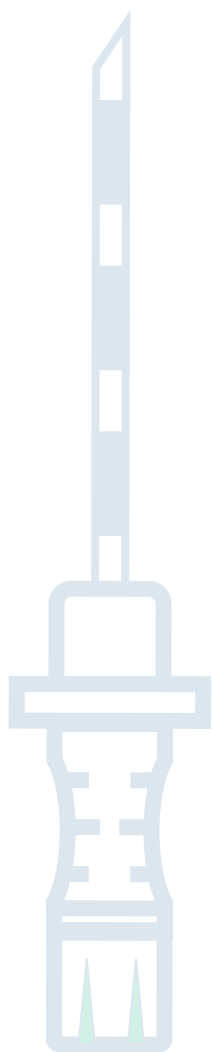


To meet individual preferences,  
B. Braun offers a broad range of  
epidural cannulas, catheters and  
accessories for epidural anaesthesia.

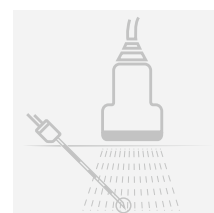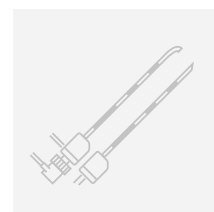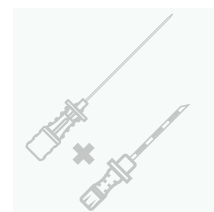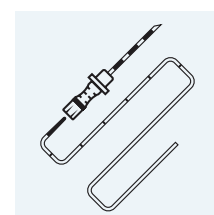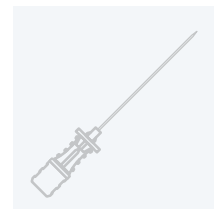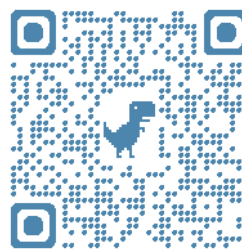

# Products for Epidural Anaesthesia


## Quality and Practicality

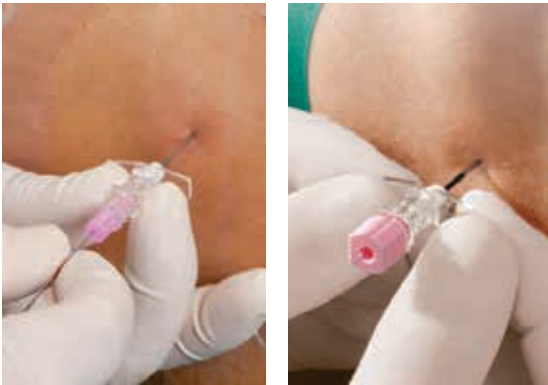

To adapt to the unique anatomical challenges of each patient, it is important to use the appropriate needle and accessories.

B. Braun focuses on

## QUALITY & PRACTICALITY.

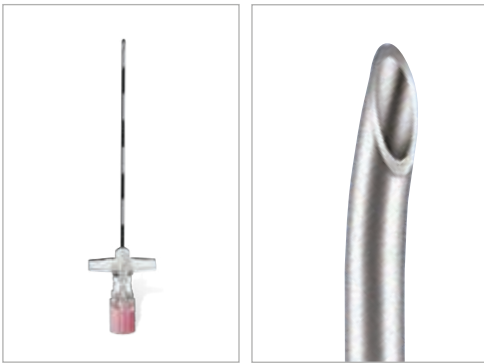

### Perican® Tuohy Needles

- For both continuous and single-shot techniques
- Different needle sizes (16 G, 18 G, 20 G)
- Polished inner bevel edge to reduce the risk of catheter shearing

#### USER BENEFITS:

- 1 cm marking to control the insertion depth
- High bending stiffness for good control during needle advancement
- Fitted stylet: to reduce the risk of tissue coring and potential epidermoid tumor development

## Perifix® Catheters: Available with different characteristics – Soft-Tip and ONE



### Perifix® ONE

- Epidural catheter with a relatively stiff but flexible shaft and a soft, atraumatic tip
- Length 1000 mm
- Six micro-holes enhance the spread of anesthetic drug
- Three yellow barium sulphate stripes allowing x-ray control
- Does not contain latex, DEHP or PVC
- MRI suitable




### Perifix® Soft-Tip Catheter

- Soft-Tip catheter with a superimposed tip
- Length 1000 mm
- Three micro-holes enhance the spread of anesthetic drug
- Clear blue length markings embedded in catheter material
- Does not contain latex, DEHP or PVC
- MRI suitable

# Products for Epidural Anaesthesia

## Perifix® Accessories

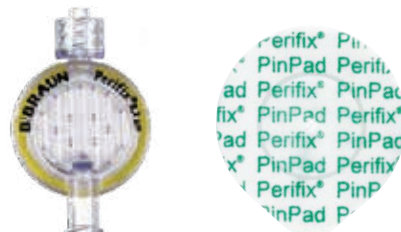

### Perifix® LOR-Syringe

- Easy-gliding plunger with good gliding characteristics and safety stop
- Parabolic graduation to differentiate it from conventional syringes
- Suitable for air and saline applications
- Lock or slip connection



### Perifix® Catheter Connector

- Available in two sizes (19 G and 20 / 24 G)
- Tactile and acoustic feedback upon closure
- Viewing window allows for catheter insertion control to ensure a successful connection

### Perifix® Filter with PinPad

- Filter fixation on patient in combination with self-adhesive PinPad
- 0.2 micron filter reduces the risk of bacterial infection
- Pressure resistant up to 7 bar



### Perifix® Catheter Fixation

- Catheter fixation, consisting of fixation ring and non-woven cover foil
- Easy examination of puncture site due to exchangeable cover foils

# Products for Epidural Anaesthesia

## Epidural Anaesthesia for the smallest patients



### Perican® Paed Epidural Needles

- Tuohy bevel
- For continuous or single-shot techniques
- Three different needle sizes (18 G, 20 G, 22 G)
- 5 mm markings to control the insertion depth
- Fitted stylets: to reduce the risk of tissue coring and potential epidermoid tumor development



### Epican® Paed – Caudal Needles

- 45° Crawford type bevel for caudal pediatric anesthesia
- Three different needle sizes (20 G, 22 G, 25 G)
- Special length (30 mm, 35 mm, 50 mm)
- 5 mm markings to control the insertion depth
- Fitted stylets: to reduce the risk of tissue coring and potential epidermoid tumor development

















### Perifix® ONE Paed Epidural Sets


Perifix® ONE Paed epidural sets in 24 G for newborns and infants.

- Catheters with distance markings for pediatric use
- For safe catheter insertion and position control
- Catheter with markings every cm in the space between 2 and 20 cm from the needle tip

# Portfolio Overview






## EPIDURAL NEEDLES






Perican®	Color code	Size (G)/Length (inch)	ø/Length (mm)	Code no. (REF) Luer
 		18 x 3 1/4"	1.30 x 80	4512383
		16 x 3 1/4"	1.70 x 80	4512782
	Long Length			
		18 x 6"	1.30 x 150	4512200
Perican® Paed	Color code	Size (G)/Length (inch)	ø/Length (mm)	Code no. (REF) Luer
 		22 x 2"	0.73 x 50	4502078
		20 x 2"	0.90 x 50	4502094
Epican® Paed Caudal	Color code	Size (G)/Length (inch)	ø/Length (mm)	Code no. (REF) Luer
 		25 x 1 1/8"	0.53 x 30	4502400
		22 x 1 3/8"	0.73 x 35	4502418
		20 x 2"	0.90 x 50	4502426

Perifix® Loss-of-Resistance (LOR) Syringe	Description	Code no. (REF) Luer
	Perifix® LOR Syringe, Luer-Lock	4638107


# Portfolio Overview


## SETS FOR EPIDURAL ANAESTHESIA (LUER)

<b>Perican® Tuohy epidural needles</b>											
20 G ø 0.90 mm / Length 50 mm		Perican® Tuohy epidural needle		Perifix® ONE epidural catheter, 6 lateral eyes		Perifix® epidural catheter, 3 lateral eyes		3 lateral eyes		Perifix® Loss-of-Resistance (LOR) syringe, Luer-Slip	
18 G ø 1.30 mm / Length 80 mm											
18 G ø 1.30 mm / Length 50 mm (Paed Set)											
16 G ø 1.70 mm / Length 80 mm											
		50/80 mm		24 G		20 G		19 G		20 G	
Complete Sets		Type		Perican®		Perifix® ONE		Perifix® Standard		Perifix® Soft Tip	







					<b>Sterican® needles</b> 25 G    Length 16 mm, ø 0,50 mm 21 G    Length 40 mm, ø 0,80 mm 18 G    Length 40 mm, ø 1,20 mm
Omnifix® three part syringe, 3 ml, Luer-Lock	Omnifix® three part syringes, 3 ml, 20 ml, Luer-Lock, 3 hypodermic needles*	Perifix® epidural filter 0.20 µm pressure resistant up to 7 bar	Perifix® PinPad filter fixation	Perifix® catheter connector	
Omnifix®	Omnifix®/Sterican®	Perifix® Filter	Perifix® PinPad	Perifix® catheter connector	Code no. (REF) Luer
	■	■	■	■	4514319
	■	■	■	■	4510305
		■		■	4510097
		■	■	■	4517309
		■	■	■	4517504
■		■	■	■	4512014C
				■	4513010
				■	4513029

# COMBINED SPINAL AND EPIDURAL ANAESTHESIA

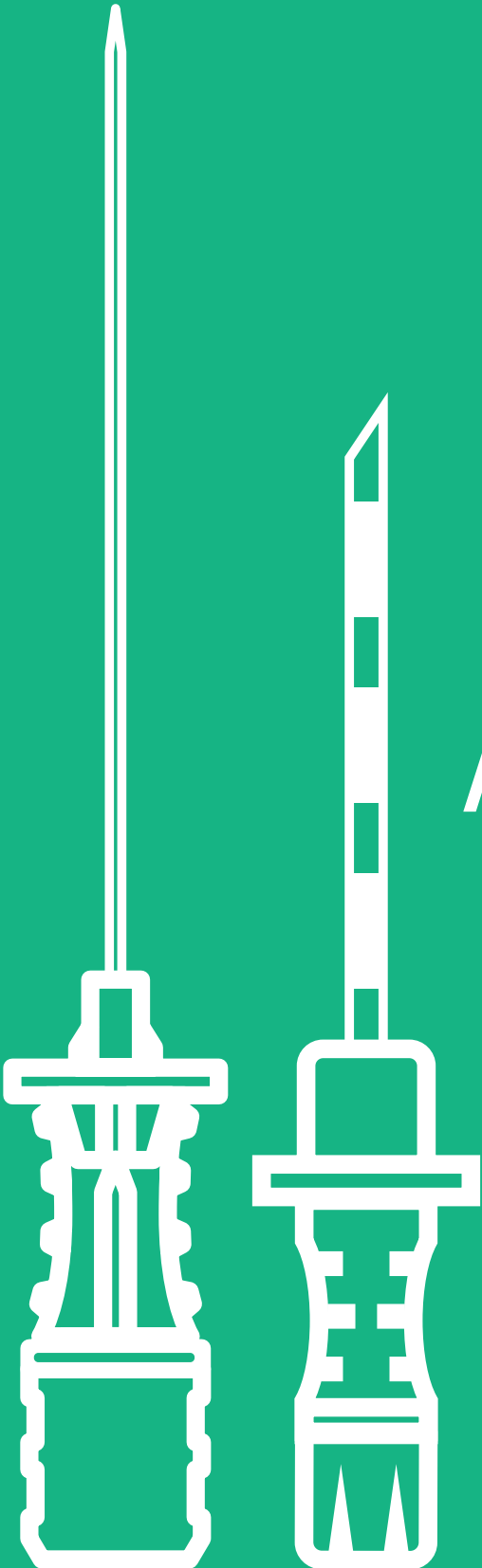

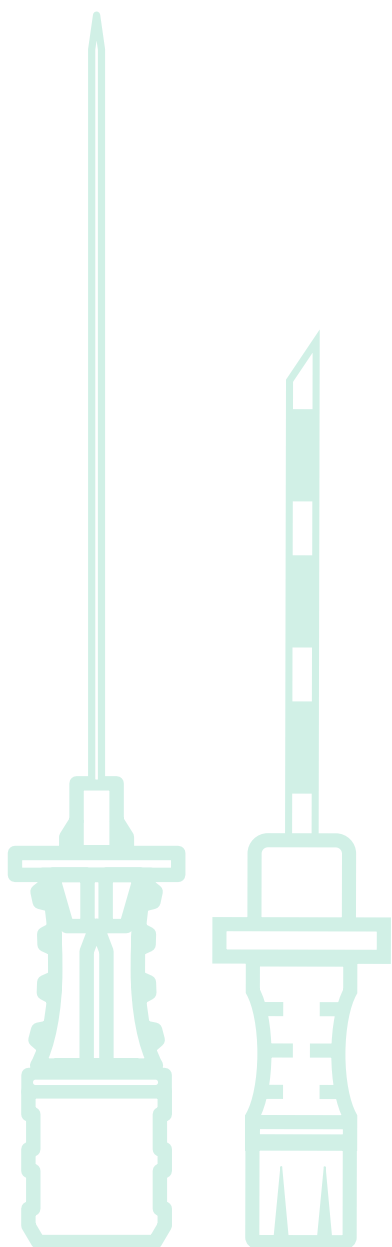


Combining the benefits of both techniques – sets for combined spinal and epidural anaesthesia (CSE).

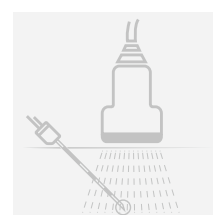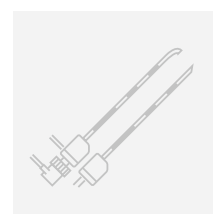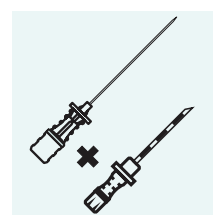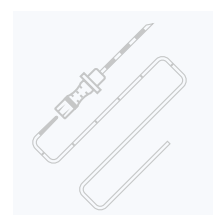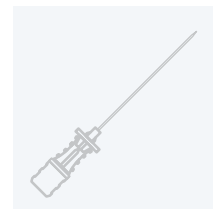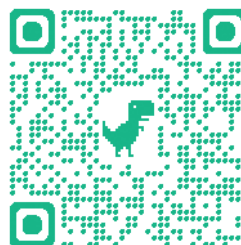

COMBINED SPINAL AND  
EPIDURAL ANAESTHESIA

# Products for Combined Spinal and Epidural Anaesthesia

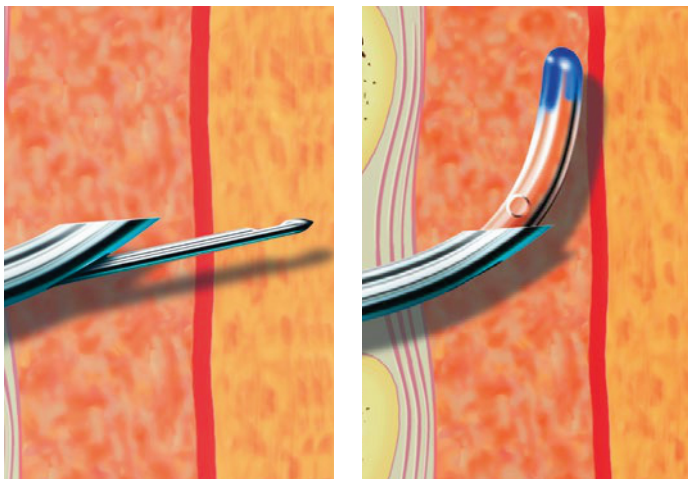

Combined Spinal-Epidural Anaesthesia (CSE):

## THE BENEFITS



of Spinal and Epidural Anaesthesia  
in the Espocan® Sets

**Special Tuohy Needle with a back-eye in the curved tip and a Spinal Needle with a plastic centering sleeve**

- Back-eye in tuohy needle for insertion of spinal needle
- Docking system to fixate the spinal needle
- Sleeved spinal needle for smooth exit through the back-eye

The ideal overlap of the spinal needle is by

Pencil Point Needle 15 mm

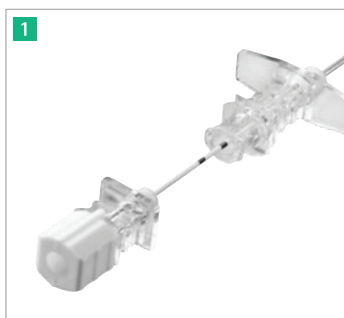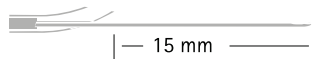

### Markings on the Spinal Needle 1

The markings on the spinal needle show when the tip leaves the back-eye.




### USER BENEFITS 2


- The docking system allows rotation of the spinal needle even in the fixed position. Avoids the difficult and uncomfortable finger grip required to steady and fix the spinal needle in position
- The docking system ensures fixation of the spinal needle inside the tuohy needle after successful dura puncture and injection of the local anesthetic


# Portfolio Overview

Espocan® and Perifix® Standard with Docking System	Size (G) / Length (inch)	ø / Length (mm)	Code no. (REF) Luer
	Pencil-Point Cannula: Pencan®		
	27 x 5 3⁄8"	0.42 x 138.5*	4556763
	All sets contain <ul style="list-style-type: none"> <li>▪ Perifix® Soft-Tip catheter, 3 lateral eyes</li> <li>▪ Docking system</li> <li>▪ Perican® Epidural Cannula 18G x 90mm</li> <li>▪ Perifix® LOR-syringe, Luer-Slip</li> <li>▪ Perifix® epidural flat filter 0.20 µm</li> <li>▪ Perifix® PinPad</li> <li>▪ Perifix® catheter connector</li> </ul>		





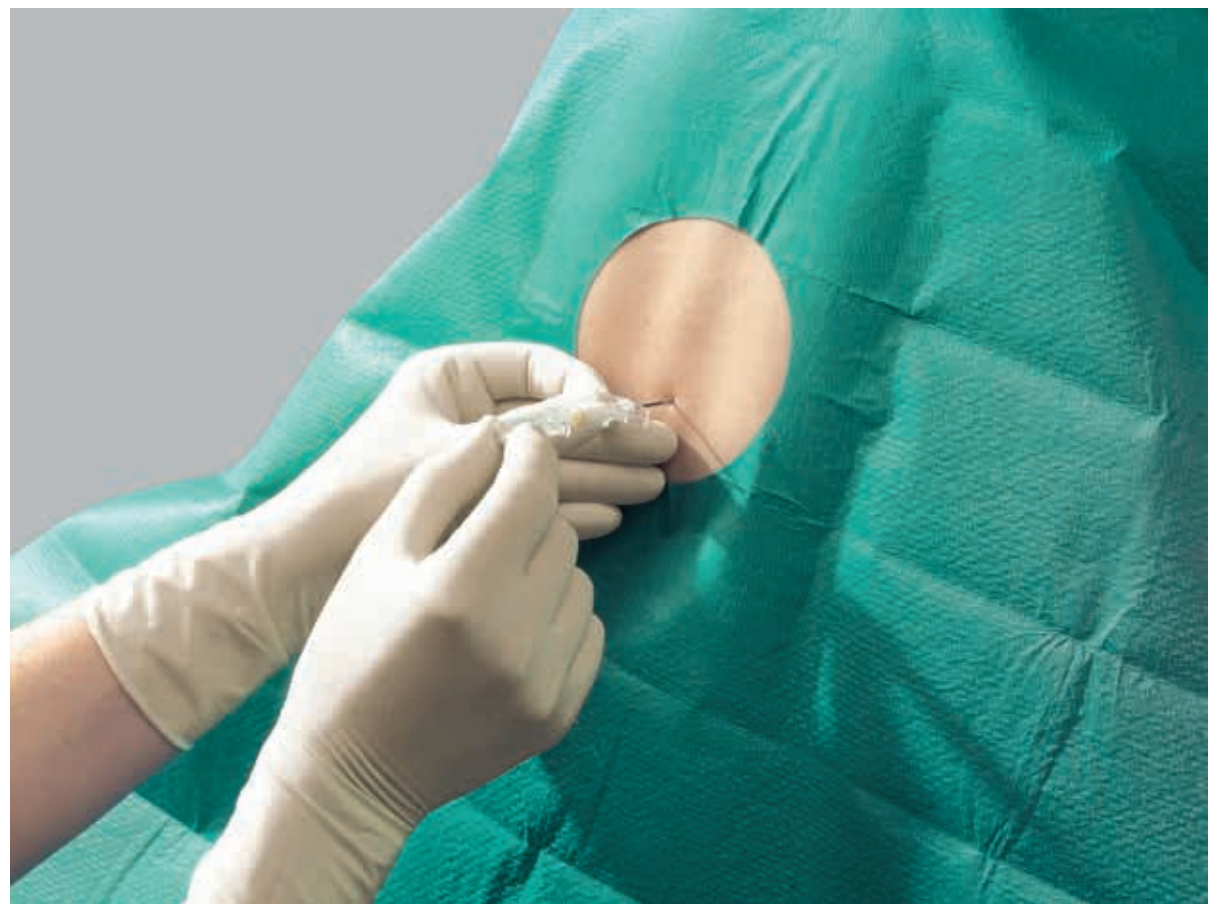


# PERIPHERAL NERVE BLOCKS

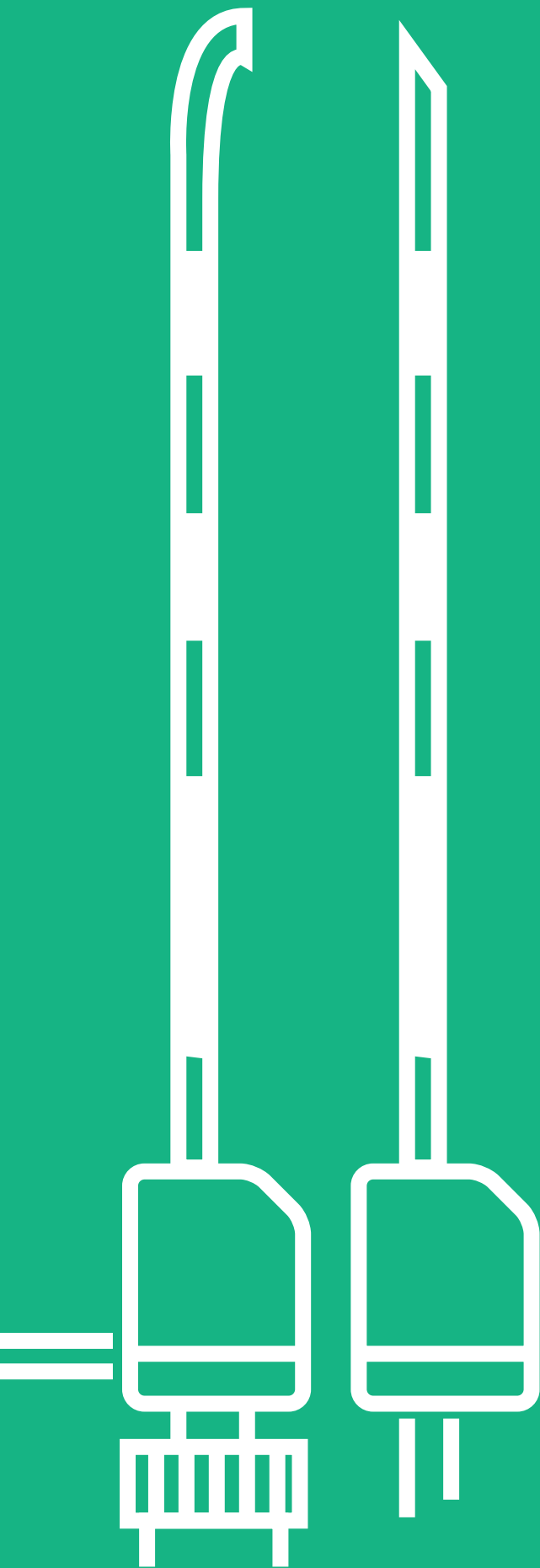

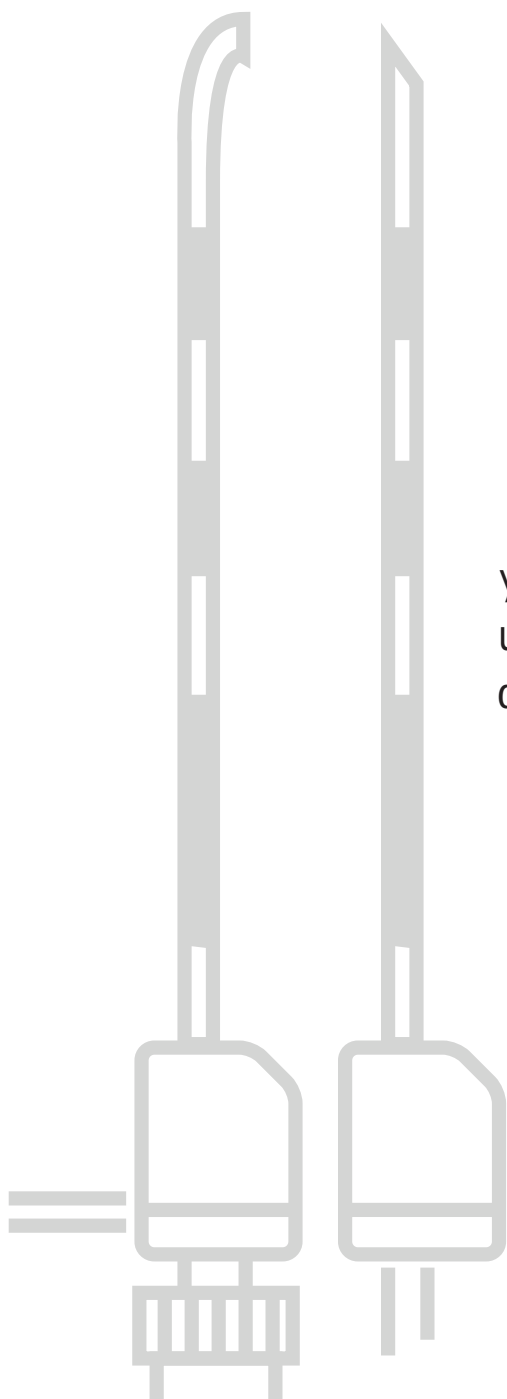


Whether you prefer to perform your blocks with nerve stimulation, under ultrasound guidance, or with dual guidance; we have the perfect product for you.

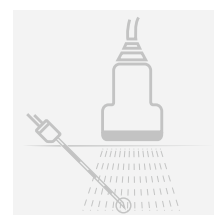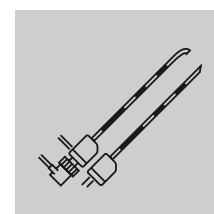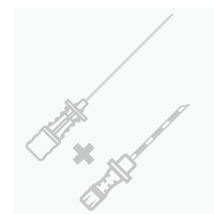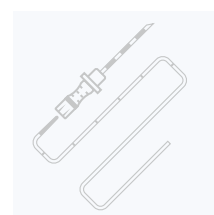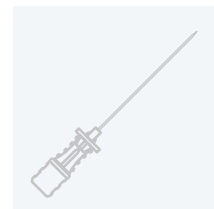

PERIPHERAL NERVE  
BLOCKS

# Peripheral Nerve Block Portfolio

Quality and functionality without compromise

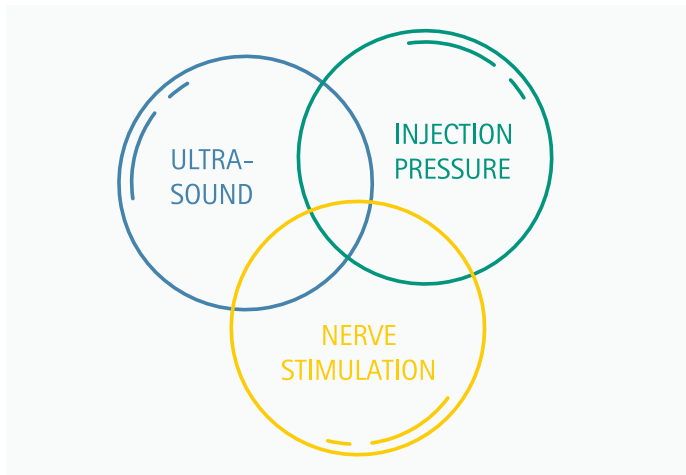

When performing a peripheral nerve block, there is no such thing as being too safe. Peripheral nerve blocks can be performed using different techniques and approaches. Electrical nerve stimulation, Ultrasound guidance and injection pressure monitoring can be used alone or in combination.

B. Braun's broad peripheral nerve block portfolio offers solutions for procedures that require ultrasound-only, nerve stimulation or dual guidance.

## ULTRASOUND

B. Braun offers a large product range of echogenic needles and catheters. Ultraline 360 is the latest generation of Ultrasound needles that combines enhanced Ultrasound visibility with smooth gliding properties and a good puncture quality.<sup>11</sup>



## NERVE STIMULATION

For electrical nerve stimulation B. Braun provides a wide range of stimulating needle and catheter systems that address the users' needs. The well established nerve stimulator Stimuplex® HNS 12 completes the offer.



## INJECTION PRESSURE MONITORING


The disposable BSmart™ pressure monitor device objectively indicates the opening injection pressure in real time and allows for a more informed decision on how to proceed with an injection regardless of who performs the block. Using this technique can prevent injection into poorly compliant tissue (e. g. nerve fascicles).

# Ultraline 360

Visibility without compromise – the next generation of Echogenic Nerve Block Needles



Circular (360°)  
X-pattern-design

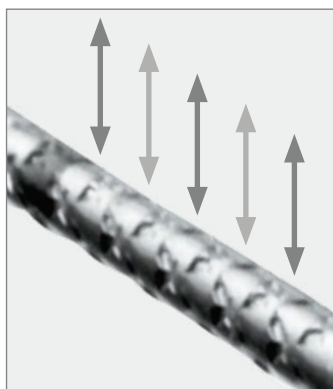


Reflection of the  
Ultrasound waves

## B. BRAUN ULTRASOUND NEEDLES – ULTRALINE 360:

The Ultraline 360 is B. Braun's latest generation of echogenic nerve block needles, featuring B. Braun's new echogenic 360° X-pattern. The 'X' is a unique shape, which when combined with a clear coating, is intended to reflect more Ultrasound waves back to the probe than previous designs.<sup>11</sup>

The X-pattern is applied as a segmented safety-code (long-short-short-tip) and facilitates needle tip identification. The Ultraline 360 portfolio combines increased Ultrasound visibility with excellent tactile feedback and puncture characteristics.

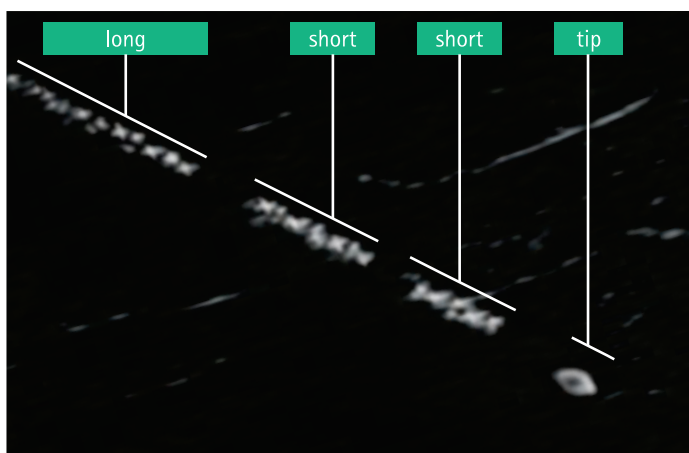

## NAVIGATION

For easy identification of the needle tip, the X-pattern is strategically applied on the first 20 mm of the needle shaft to create the segmented code "long-short-short-tip". This enables distinction between the needle tip and the upper needle shaft.<sup>11</sup>

## USER BENEFITS

- **Navigation:** Safety code for identification of the needle and needle tip position
- **Visibility:** 360° X-pattern and clear coating result in better Ultrasound visualization and tip identification compared to previous designs
- **Tactile Feedback:** designed for consistent puncture – same 30° back-cut bevel as previous Stimuplex® Ultra and Ultraplex® needles

# Products for Peripheral Nerve Blocks








## Single Shot Needles

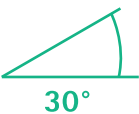


Stimuplex® and Ultraplex® single shot needles offer precision and ergonomomy and are designed for high success rates.

### USER BENEFITS

- Excellent puncture and gliding characteristics
- Broad portfolio of needle lengths from 50 to 150 mm and G 22 to G 20
- Ergonomic needle hub design for enhanced tactile feedback

Cannula	Bevel	Insulation	Echogenic properties	Stimulating capabilities
	 30°	 Full Tip		
Stimuplex® A 	✓	✓		✓
Stimuplex® Ultra 360® 	✓	✓	✓	✓
Ultraplex® 360 	✓		✓	

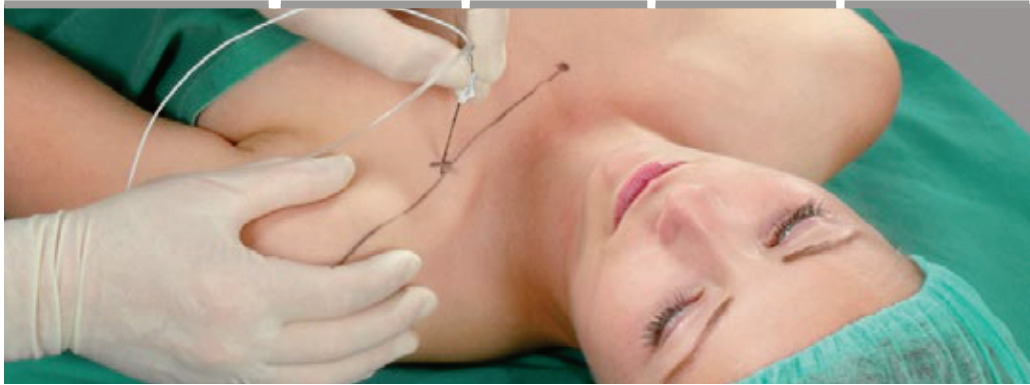

# Products for Peripheral Nerve Blocks

## Sets for Continuous Peripheral Nerve Blocks










### Contiplex® Continuous Nerve Block Sets

Continuous peripheral nerve blocks provide pain relief during and after surgical procedures, for improved patient recovery.<sup>12</sup>

#### USER BENEFITS

- Wide range of catheter sets
- Harmonized set components
- Good Ultrasound visibility of Perifix® ONE catheter

Set	Bevel		Echogenic properties	Catheter
	 15°	 20°	 Tuohy	 ONE
Catheter through the needle technique				
Contiplex® S Ultra 360® 		✓		✓
Contiplex® Tuohy Ultra 360® 			✓	✓
Catheter over the needle				
Contiplex® C 	✓		✓	✓

# Products for Peripheral Nerve Blocks

## Nerve Stimulation with the Stimuplex® HNS 12



1 Stimuplex® HNS 12 with SENSE



2 Stimuplex® Pen for cutaneous nerve mapping


### B. Braun Nerve Stimulator – Stimuplex® HNS 12 with SENSE

#### What is SENSE?


In practice, marginal needle movements often lead to a loss of muscle twitch. The SENSE technology (Sequential electrical Nerve Stimulation) is a sequence of two fixed impulse durations of 0.1 ms along with a third longer impulse duration.<sup>13</sup> Most traditional nerve stimulators emit two very short pulses every second. That's because short pulses improve accuracy when closer to the nerve. The downside is that sensitivity, or the ability to elicit visual feedback at a distance from the nerve, is lost. SENSE technology increases both sensitivity and accuracy by adding a third, longer pulse. This results in eliciting a motor response even when the needle is far from, or not aimed directly at the nerve, and requires less adjustment to the amperage control while retaining the visual feedback.

#### USER BENEFITS


- Stimulation with SENSE-technology designed for more efficient nerve stimulation<sup>13</sup>
- Menus designed for easy control of sound and other settings
- Acoustic signals inform the user of any relevant deviations

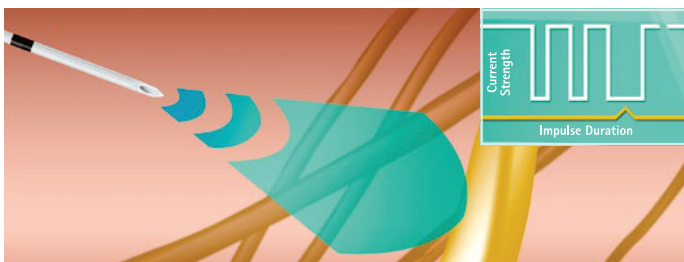

As the needle is advanced, SENSE generates two short and one long pulse. From the alternating sequence of three impulses, the third longer impulse stimulates the nerve from a distance. One muscle twitch is visible and indicates that the needle is directed to the nerve.

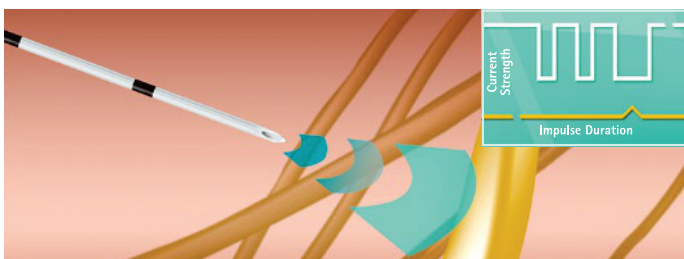

The stimulation needle is further advanced towards the nerve while SENSE maintains the motor response. Current strength adjustments lead to definite reductions of the third impulse.

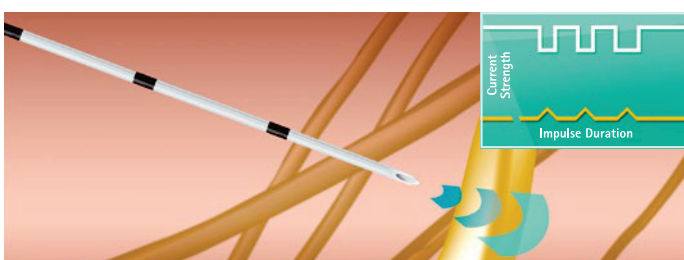

At the threshold level of approximately 0.5 mA three muscle twitches are visible as all three impulses stimulate the nerve.

# Products for Peripheral Nerve Blocks

## Injection Pressure Monitoring with BSmart™






### Injection Pressure Monitoring

Accidental injection of local anesthetic into fascicles of peripheral nerves can result in permanent nerve injury.<sup>1</sup> Research has shown that high opening injection pressure (> 15 psi) indicates needle-nerve contact or intrafascicular injection.<sup>14, 15, 16, 17, 18</sup>

### USER BENEFITS

- Objectively measures and displays injection pressure
- Color coding visualizes the pressure information
- Allows consistent monitoring of resistance to injection regardless of who performs the actual injection<sup>19</sup>
- Allows for standardized and objective documentation of injection pressure information during nerve block procedures




   The pressure information is conveniently displayed by the colour-coded pressure ranges on the BSmart™ piston.


### BSmart™ Injection Pressure Monitor


The BSmart™ Injection Pressure Monitor is an easy-to-use in-line injection pressure monitoring device providing clinicians with objective pressure information. This happens through a coloured piston which climbs depending on the pressure. BSmart™ Injection Pressure Monitor does not interfere with the tactile feedback of the syringe which has been traditionally used to assess the resistance of injection during nerve block procedures.

# Portfolio Overview


## SINGLE-SHOT BLOCK NEEDLES

Stimuplex® A	Needle ø (G) and Length (inch)	Needle ø and Length (mm)	Units per box	Code no. (REF) Luer
	Needles with 30° bevel			
	24 x 1"	0.55 x 25 mm	25	4894251
	22 x 2"	0.70 x 50 mm		4894502
	21 x 4"	0.80 x 100 mm		4894260
	20 x 6"	0.90 x 150 mm		4894278

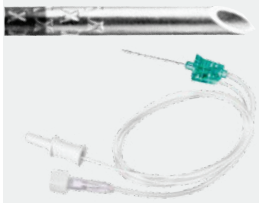Stimuplex® Ultra 360®	Needle ø (G) and Length (inch)	Needle ø and Length (mm)	Units per box	Code no. (REF) Luer
	Insulated Echogenic Needles with 30° Bevel			
	22 x 2"	0.70 x 50 mm	25	4892505-01
	22 x 3 1/8"	0.70 x 80 mm		4892508-01
	20 x 4"	0.90 x 100 mm		4892510-01
	20 x 6"	0.90 x 150 mm		4892515-01

Ultrplex® 360	Needle ø (G) and Length (inch)	Needle ø and Length (mm)	Units per box	Code no. (REF) Luer
	Non-Insulated Echogenic Needles with 30° Bevel			
	22 x 2"	0.70 x 50 mm	25	4892605-01
	22 x 3 1/8"	0.70 x 80 mm		4892608-01
	20 x 4"	0.90 x 100 mm		4892610-01
	20 x 6"	0.90 x 150 mm		4892615-01

BSmart™	Description	Code no. (REF)
	BSmart™ Injection Pressure Monitor	4894000-01

Nerve Stimulation	Description	Code no. (REF)
	Stimuplex® HNS 12 Nerve Stimulator for Stimuplex®, Contiplex® needles	4892098

Nerve Stimulator Accessories and Spare Parts	Code no. (REF)
Stimuplex® Pen for cutaneous nerve mapping	4892099
Electrode cable for Stimuplex® HNS 11/12	4892070
Battery cover for Stimuplex® HNS 12	4892094
Carrying Case for Stimuplex® HNS 12	4892095
Test resistor for Stimuplex® HNS 12	4892096
Knob for special handling	4892283

\*






# Portfolio Overview

## SETS FOR CONTINUOUS NERVE BLOCKS


		
Contiplex® C catheter-over- the-needle system	Contiplex® S Ultra 360®	Contiplex® Tuohy Ultra 360®
25 G	18 G	18 G

Set Specifications	Bevel (°)	Needle		
		Size (G)	Size Ø (mm)	Length (mm)
Contiplex® C Sets				
Contiplex® C	15	25	0.53	190
Contiplex® S Ultra 360® Sets				
Contiplex® S Ultra 360® 50 mm	20	18	1.30	50
Contiplex® S Ultra 360® 100 mm	20	18	1.30	100
Contiplex® S Ultra 360® 150 mm	20	18	1.30	150
Contiplex® Tuohy Ultra 360® Sets				
Contiplex® Tuohy Ultra 360® 50 mm	Tuohy bevel	18	1.30	50
Contiplex® Tuohy Ultra 360® 100 mm	Tuohy bevel	18	1.30	100
Contiplex® Tuohy Ultra 360® 150 mm	Tuohy bevel	18	1.30	150




						
	Contiplex® ONE Catheter (6 lateral openings, closed tip, open tip for Contiplex® C)	Perifix® Catheter Connector 19 G or 20 / 24 G	Perifix® PinPad filter fixation	Perifix® Filter 0.20 µm, Pressure resistant up to 7 bar	Omnifix® syringe 5 ml	
Set Specifications	Contiplex® ONE (mm)	Perifix® Catheter Connector	Perifix® PinPad	Perifix® Filter	Omnifix®	Code no. (REF)

Contiplex® C Sets						
Contiplex® C	187	■	■	■	■	4898115
Contiplex® S Ultra 360® Sets						
Contiplex® S Ultra 360® 50 mm	400	■	■	■	■	4898650-01
Contiplex® S Ultra 360® 100 mm	1000	■	■	■	■	4898610-01
Contiplex® S Ultra 360® 150 mm	500	■	■	■	■	4898615-01
Contiplex® Tuohy Ultra 360® Sets						
Contiplex® Tuohy Ultra 360® 50 mm	400	■	■	■	■	4898705-01
Contiplex® Tuohy Ultra 360® 100 mm	1000	■	■	■	■	4898710-01
Contiplex® Tuohy Ultra 360® 150 mm	500	■	■	■	■	4898715-01

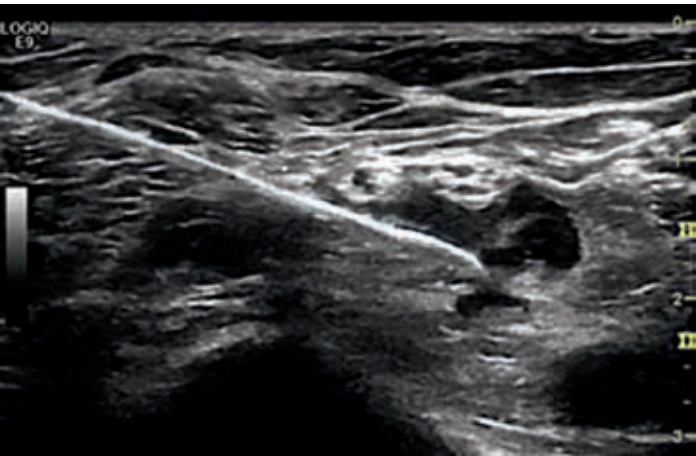

Contiplex® C catheter with cannula as seen under ultrasound

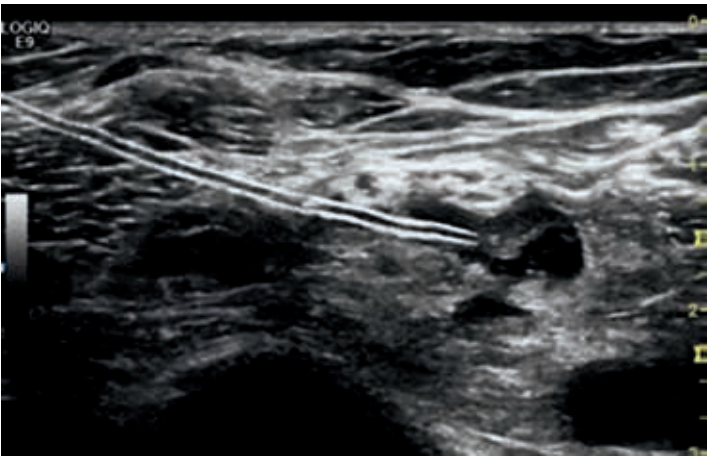

Contiplex® C catheter with cannula removed as seen under ultrasound

## LITERATURE

1. Neuman MD et al. Comparative Effectiveness of Regional versus General Anesthesia for Hip Fracture Surgery in Adults. *Anesthesiology* 2012;117(1):72-92. doi: <https://doi.org/10.1097/ALN.0b013e3182545e7c>.
2. Ilfeld BM et al. Continuous Popliteal Sciatic Nerve Block for Postoperative Pain Control at Home: A Randomized, Double-Blinded, Placebo-Controlled Study. *Anesthesiology* 2002;97(4):959-965.
3. Aguirre J. The economics of regional versus general anaesthesia in the ambulatory setting. *Euroanaesthesia: The European Anaesthesiology Congress*; 2016 May 28-30; London. <http://euroanaesthesia2016.esahq.org/the-economics-of-regional-versus-general-anaesthesia-in-the-ambulatory-setting/>.
4. Hu S et al. A comparison of regional and general anaesthesia for total replacement of the hip or knee: a meta-analysis. *J Bone Joint Surg Br* 2009; 91(7):953-42. doi: 10.1302/0301-620X.91B7.21538.
5. Helwani MA et al. Effects of regional versus general anesthesia on outcomes after total hip arthroplasty: a retrospective propensity-matched cohort study. *J Bone Joint Surg Am.* 2015; 97(3):186-93. doi: 10.2106/JBJS.N.00612.
6. ISO 80369-6:2016; Small bore connectors for liquids and gases in healthcare applications — Part 6: Connectors for neuraxial applications. <https://www.iso.org/standard/50734.html>.
7. Lehmann, 2002, Spinalkanülen neuester Generation.
8. Vallejo MC et al. Postdural puncture headache: a randomized comparison of five spinal needles in obstetric patients. *Anesth Analg.* 2000 Oct; 91(4):916-20.
9. Krommendijk et al. The PENCAN 25-gauge needle: A new pencil-point needle for spinal anesthesia tested in 1,193 patients. *Reg Anesth Pain Med.* 1999;24(1):43-50.
10. Scott DB et al. Atraucan: a new needle for spinal anesthesia. *Regional anesthesia* 1993;18(4):213-7.
11. Cooper D et al. Regional Block Needles—A Matter of Feel and Brightness. Presented at the 35th ESRA in Maastricht, Netherlands, September 2016.
12. Steinfeld T et al. Nerve injury by needle nerve perforation in regional anesthesia: does size matter? *Br J Anaesth.* 2010;104(2):245-53. doi: 10.1093/bja/aep366.
13. Urmei WF, Grossi P. Use of sequential electrical nerve stimuli (SENS) for location of the sciatic nerve and lumbar plexus. *Reg Anesth Pain Med.* 2006;31(5):463-9.
14. Farber SJ et al. Peripheral nerve injury after local anesthetic injection. *Anesth Analg.* 2013; 117(3):731-9. doi: 10.1213/ANE.0b013e3182a00767.
15. Gadsden et al. Opening injection pressure consistently detects needle-nerve contact during ultrasound-guided interscalene brachial plexus block. *Anesthesiology.* 2014;120(5):1246-53. doi: 10.1097/ALN.000000000000133.
16. Kapur E et al. Neurologic and histologic outcome after intraneural injections of lidocaine in canine sciatic nerves. *Acta Anaesthesiol Scand.* 2007; 51(1):101-7.
17. Orebaugh SL et al. Brachial plexus root injection in a human cadaver model: Injectate distribution and effects on the neuraxis. *Reg Anesth Pain Med.* 2012; 37(5):525-9.
18. Bigeleisen PE et al. Extraneural versus intraneural stimulation thresholds during ultrasound-guided supraclavicular block. *Anesthesiology.* 2009;110(6):1235-43. doi: 10.1097/ALN.0b013e3181a59891.
19. Theron PS et al. An animal model of 'syringe feel' during peripheral nerve block. *Reg Anesth Pain Med.* 2009;34:330-2.
20. Kāsine T et al. Needle tip tracking for ultrasound-guided peripheral nerve block procedures—An observer blinded, randomised, controlled, crossover study on a phantom model. *Acta Anaesthesiol Scand.* 2019; 63(8): 1055-62. doi: 10.1111/aas.13379.
21. Kāsine T et al. The effect of needle tip tracking on procedural time of ultrasound-guided lumbar plexus block: a randomised controlled trial. *Anaesthesia* 2019. 75(1): 72-79. doi: 10.1111/anae.14846.
22. McLeod GA et al. An initial evaluation of the effect of a novel regional block needle with tip-tracking technology on the novice performance of cadaveric ultrasound-guided sciatic nerve block. *Anaesthesia* 2020;75(1):80-88. doi: 10.1111/anae.14851.
23. 2019-04-06\_Customer feedback\_Onvision performance questionnaire\_D000228286 CER.
24. Liu SS et al. Efficacy of continuous wound catheters delivering local anesthetics for postoperative analgesia: a quantitative and qualitative systemic review of randomized controlled trials. *J Am Coll Surg.* 2006;203(6):914-932. doi: 10.1016/j.jamcollsurg.2006.08.007.
25. Chely JE et al. Continuous peripheral nerve blocks in acute pain management. *Br J Anaesth.* 2010; 105(1): 186-196. doi: 10.1093/bja/aeq322.
26. Ilfeld B, Continuous peripheral nerve blocks: A review of the published evidence. *Anesth Analg* 2011; 113(4):904-925. doi: 10.1213/ANE.0b013e3182285e01.
27. Bingham A et al. Continuous peripheral nerve block compared with single-injection peripheral block. A systematic review and meta-analysis of randomized controlled trials. *Reg Anesth Pain Med.* 2012;37(6):583-94. doi: 10.1097/AAP.0b013e31826c351b.
28. Bharti N et al. The efficacy of a novel approach to transverse abdominis plane block for postoperative analgesia after colorectal surgery. *Anesth Analg.* 2011;112(6):1504-8. doi: 10.1213/ANE.0b013e3182159bf8.
29. Belavy D et al. Ultrasound-guided transverse abdominis plane block for analgesia after caesarean delivery. *Br J Anaesth.* 2009;103(5):726-30. doi: 10.1093/bja/aep235.
30. Beaussier M et al. Continuous preperitoneal infusion of ropivacaine provides effective analgesia and accelerates recovery after colorectal surgery. *Anesthesiology.* 2007; 107(3):461-468.